

# LeucoScreen

Document ID: FP09 I05 R01 B.12 Update: 24/05/2018

Semi-quantitative histochemical kit for the determination of peroxidase-positive white blood cells in semen

For in vitro diagnostic use - Reagent for professional use only



via link on our website, or scan barcode):



## CALCULATION OF THE CONCENTRATION OF PEROXIDASE-POSITIVE WHITE BLOOD CELLS

1. Calculate the proportion of peroxidase-positive cells as follows:

PROPORTION POSITIVE ROUND CELLS =

$$\frac{\text{Number of POSITIVE round cells}}{(\text{Number of POSITIVE round cells} + \text{Number of NEGATIVE round cells})}$$

2. Now, calculate the concentration of peroxidase-positive white blood cells in the semen sample as follows:

CONCENTRATION (mill/mL) =

Proportion positive round cells x total concentration of round cells

Example:

- Total concentration of round cells is 2 mill/mL (determined during sperm concentration analysis)
- With the LeucoScreen test, 120 round cells are found positive and 80 round cells are found negative
- Proportion positive round cells =  $\frac{120}{(120+80)} = 0.6$
- Concentration of peroxidase-positive white blood cells =  $0.6 \times 2$  mill/mL = 1.2 mill/mL

## STORAGE

Suitable for transport or short term storage at elevated temperatures (up to 5 days at 37°C). Store reagents between 2°C-25°C.

## REMARKS

Formation of a sediment in Reagent 1 is normal. Simply pour Reagent 1 over filter-paper to eliminate sediment.

## LIMITATIONS OF THE METHOD

LeucoScreen only stains peroxidase-positive WBC, other types of WBC (e.g. lymphocytes and monocytes) cannot be detected.

## SENSITIVITY AND SPECIFICITY

Sensitivity and specificity for leukocytospermia is 90% when compared with the immunohistological test<sup>8</sup>, with a threshold for the peroxidase stain of 1 mill. WBC/ml and for the immunohistological test of 2 mill. WBC/ml.

## WARNINGS AND PRECAUTIONS

All semen samples should be considered potentially infectious. Handle all specimens as if capable of transmitting HIV or hepatitis. Reagent 1 is very poisonous by inhalation, skin contact or swallowing. Risk of unrepairable damage. Wear protective clothing and take off contaminated clothing immediately. Work under a fume hood. In case of any accident, seek medical attention. Reagent 2 is corrosive and causes burns. After contact with skin wash immediately with water and soap. Wear eye / face protection.

## BIBLIOGRAPHY

1. Wolff, H., Anderson, D.J. (1988) Immunohistological characterization and quantification of leukocyte subpopulation in human semen. Fertility and Sterility, 53:528-36.
2. Aitken, R.J., West, K.M. (1990) Analysis of the relationship between reactive oxygen species production and leukocyte infiltration in fractions of human semen separated on Percoll gradients. International Journal of Andrology, 13:433-51.
3. Barratt, C.L.R., Bolton, A.E., Cooke, I.D. (1990) Functional significance of white blood cells in the male and female reproductive tract. Human Reproduction, 5:639-44.
4. WHO laboratory manual for the examination and processing of human semen, 5th edition (2010), p. 102-107.
5. Aitken, R.J., Clarkson, J.S., Fishel, S. (1989) Generation of reactive oxygen species, lipid peroxidation and human sperm function. Biology of Reproduction, 41:183-7.
6. Hill, J.A., Haimovici, F., Politch, J.A., Anderson, D.J. (1987) Effects of soluble products of activated lymphocytes and macrophages (lymphokines and monokines) on human sperm motion parameters. Fertility and Sterility, 47:460-5.
7. Endtz, A.W. (1972) Een methode om het vochtige urinesediment en het vochtige menselijke sperma rechtstreeks te kleuren. Nederlands Tijdschrift voor Geneeskunde, 116(17): 681-5.
8. Politch, J.A., et al (1993) Comparison of methods to enumerate white blood cells in semen. Fertility and Sterility, 60(2): 372-5.



FertiPro N.V., Industriepark Noord 32, 8730 Beernem, Belgium. URL: <http://www.fertipro.com>  
E-mail: [info@fertipro.com](mailto:info@fertipro.com)

REF

LEUCO



We recommend to view our demonstration video (download