Sil-Select Stock™

EN

STERILE A

Document Id: FP09 I13 R01 D.1
Mise à jour: 15.11.2019

USED ABBREVIATIONS

EBSS Earle's Balanced Salt Solution
ICSI Intra-uterine Inoculation
FIV In Vitro Fertilization
IUI Intra-Uterine Insemination

INTENDED USE AND COMPOSITION

Sil-Select Stock is a stock solution for semen preparation. It is a ready-to-use isotonic gradient for semen preparation with a density of approximately 1.12 g/ml. Sil-Select Stock consists of saline-coated colloidal silica particles suspended in HEPEs-buffered Earle's balanced salt solution and some product codes are supplemented with 10 mg/l gentamicin. Sil-Select Stock can be used in combination with IUI, IVF and ICSI.

MATERIAL INCLUDED

Product code: SIS100
» 1x 100ml Sil-Select Stock
Product code: SIS500
» 1x 500ml Sil-Select Stock
Product code: SIS100_320
» 1x 100ml Sil-Select Stock with gentamicin
Product code: SIS500_06
» 1x 500ml Sil-Select Stock with gentamicin

MATERIAL NEEDED BUT NOT PROVIDED

» Stereocylinders with 1.12/21 g needle
» Centrifuge must be able to operate for up to 30 minutes
» 1x 400 g
» Incubator or water bath at 37°C (optional)
» Ferticall® Flushing Medium or Sil-Select Plus Washing/Insemination medium (FerPro® N.V.)
» LAF bench (ISO Class 5)

PRECAUTIONS

» Always work under strict hygienic conditions, i.e. LAF-bench ISO Class 5) to avoid possible contamination. Aseptic technique should be used to avoid post-thaw contamination, even when the products contain gentamicin.
» All human, organic material should be considered potentially infectious. Handle all specimens as if capable of transmitting HIV or hepatitis.
» Always wear protective clothing when handling specimens.

PRODUCT SPECIFICATIONS

» Chemical composition
» pH value criteria: 7.20-7.90
» pH stability criteria: 7.20-7.90
» Osmolality: 300-330 mOsm/kg
» Density: 1.1150 - 1.1250 g/ml
» Viscosity: < 1.75 cP
» Expiry date: 12 months
» Sterility test: by Limulus Amoebocyte Lysate methodology (LAL) <0.5 EU/ml
» Sterility test: by the culture method (Fur. 2.6.1) No growth
» Spenn Survival test: > 80% survival after 48 hours exposure of directly selected spermatozoa to the test medium
» Not for use in testis
» A certificate of analysis and MSDS are available upon request.

CALCULATIONS OF G-FORCES

The g-force of your centrifuge can be calculated using this formula:
$$g = 1.118 \times r \times \pi^2 \times rpm^2 / rpm = \text{Square root } g \text{ (1.118 x r)}$$

r = radius of centrifuge in mm
rpm = rotations per minute / 1000
Example 1
r = 100 mm
rpm = 1800 rotations per minute
g = 1.118 x 100 x 3.24 = 362g
Example 2
r = 100 mm
rpm = 50R (50 / (1.118 x 100)) = 1.77
rpm = 1770 rotations per minute

WARNINGS BEFORE USE

- Do not use the product if:
 - It becomes cloudy or shows any evidence of microbial contamination
 - It is outside of the container or shows any defect when the product is delivered
 - Expiry date has been exceeded
 - Do not freeze before use
 - Do not re-sterilize after opening
 - Products that include gentamicin should not be used on a patient that has a known allergy to gentamicin or similar antibiotics

INSTRUCTIONS FOR PREPARATION OF GRADIENTS

We advise the gradient bottles by 5 bottle inversions before use.

You must prepare a dual gradient system (45% - 90% or 40% - 80%) starting from Sil-Select Stock. If preferred a multi-layer can be used as well (e.g. 45% - 70% - 80%). To prepare a 90% gradient, mix 1 part Ferticall® Flushing medium (or Sil-Select Plus Washing/Insemination medium) with 9 parts Sil-Select Stock containing 0.12 g/ml gentamicin. A 45% gradient is prepared by mixing 5.5 parts Ferticall® Flushing medium (or Sil-Select Plus Washing/Insemination medium) with 4.5 parts Sil-Select Stock. Alternatively any HEPEs-buffered EBSS-based medium can be used for the preparation of the gradients.

Note: Gradients should be prepared and repeated under sterile conditions (e.g. LAF bench ISO Class 5). For optimal results, prepare the gradients within 18 hours prior to use. Store at 2-8°C and warm gradients to room temperature or 37°C one hour before use. Mix well after diluting the Sil-Select Stock solution. Follow the instructions as indicated for Sil-Select Plus in order to use the prepared density gradient media for sperm selection.

STORAGE AND CONSERVATION

- Store products between 2-8°C
- The products can be used up to 7 days after opening, when sterile conditions are maintained and the products are stored at 2-8°C.
- Keep away from sunlight.
- The products are stable after transport (max. 5 days) at elevated temperatures (< 37°C).

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PRECAUTIONS

- Always work under strict hygienic conditions, i.e. LAF-bench ISO Class 5) to avoid possible contamination. Aseptic technique should be used to avoid post-thaw contamination, even when the products contain gentamicin.
- All human, organic material should be considered potentially infectious. Handle all specimens as if capable of transmitting HIV or hepatitis.
- Always wear protective clothing when handling specimens.

CALCULATIONS OF G-FORCES

The g-force of your centrifuge can be calculated using this formula:

$$g = 1.118 \times r \times \pi^2 \times rpm^2 / rpm = \text{Square root } g \text{ (1.118 x r)}$$

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