



OOSAFE[®] MEDICAL DEVICE DISINFECTANT

CLASS IIA MEDICAL DEVICE DISINFECTANT, CE MARKED IN ACCORDANCE WITH EUROPEAN MEDICAL DEVICE DIRECTIVE 93/42/EEC

Manufactured in an EN ISO 13485:2016 certified facility.



GOosafe in IVF Laboratories

OOSAFE® MEDICAL DEVICE DISINFECTANT



Health and safety

- MEA and HSSA tested
- Transparent
- Odorless
- Alcohol-free
- No VOC release

High quality

- Ready-to-use solution
- Anticorrosive
- Securely sealed
- Valid for 3 years, even after opening
- Long term disinfection/low evaporation rate
- Fast disinfection effect

Directions:

Spray: Apply undiluted, spray and distribute evenly on the surface with a sterile cloth. Wait 15 minutes for the disinfection effect.

Wipes: Wipe the surface evenly and wait 15 minutes for the disinfection effect.

Manufactured in an EN ISO 13485:2016 certified facility.

BACTERICIDAL

Pseudomonas aeruginosa

15 min Staphylococcus aureus
Enterococcus hirae

FUNGICIDAL

15 min Candida albicans

MYCOBACTERICIDAL AND TUBERCULOCIDAL

15 min Mycobacterium terrae
Mycobacterium avium

VIRUCIDAL

15 min Vaccinia virus*
Poliovirus
5 min Adenovirus

*Effective against all enveloped viruses including HIV, Hepatitis B and C, Herpes simplex, Influenza A, Ebola

TIME UNTIL FULL EFFECT

MICROORGANISM



Product code:
OOMDW-120
Oosafe® Disinfectant - Wipes,
70 wipes per bucket.



Product code:
OOMD-01000
Oosafe® Disinfectant,
1 Liter bottle with spray



Product code:
OOMD-05000
Oosafe® Disinfectant,
5 Liters refill container



Product code:
OOMD-00050
Oosafe® Disinfectant,
50 ml in spray bottle



facebook.com/sparMED



twitter.com/sparMED



linkedin.com/company/sparmed-aps

